

FAST-SET

Versatile & Fast-Setting for Minimal Downtime

A fast-curing, urethane-based adhesive for crack repairs, hoof extensions, and custom shoes.



Engineered for farriers who need a dependable, easy-to-use solution for fast hoof care.



Set Time	Base Material	Packaging	Application
30 seconds	Urethane	Durable 210mL cartridge	Rapid, durable repairs to small hoof issues and toe cracks

Farriers Choice® Fast-Set is a two-component, fast-curing, non-sag polyurethane adhesive designed for crack repairs, hoof extensions, and custom shoes. Odorless and easy to apply, it creates a strong, secure bond and sets in just 30 seconds. It's ideal for farriers who need quick, reliable performance in structural hoof care.

- **Fast & Reliable** – 30-second set time for rapid application and minimal downtime.
- **Easy Application** – Convenient, all-in-one solution with secure mixing tip to ensure precise placement.
- **Strong Bonding** – Provides superior adhesion for repairing small hoof issues, toe cracks, and extensions.
- **Customizable Use** – Ideal for foal cuffs and building custom horseshoes.
- **No Mess, No Waste** – Resealable, heavy-duty cartridge prevents leaks, preserves product, and allows multiple uses.

Resealable twist-off cap protects product between uses.

*Cap and mixing tip fit into the cartridge **one way only**—to prevent cross contamination and eliminate material waste.*

Mixing tip included with each cartridge!



Trusted by Farriers, Proven in the Field. Learn more at farrierschoice.com.

Farrier's Choice® offers a range of specialized adhesives, each tailored to address specific hoof care needs and designed to meet specific challenges faced by professional farriers, ensuring optimal hoof health and performance.



Farriers Choice® Fast-Set

Versatile and fast-setting for minimal downtime. Excellent for crack repairs, hoof extensions, and custom shoes.

Set Time: 30 seconds Base: Urethane Color: Clear

TYPICAL PROPERTIES

	Part A	Part B	Mixed
Color:	Amber	Light Amber	Light Amber
Viscosity @ 77°F, cps:	500 - 1,000	1,500 - 4,000	Paste
Weight per gallon, pounds:	9.03 - 9.43	8.35 - 8.75	-
Gel Time, [9.4 grams @ 77°F], seconds:	-	-	15 - 25
Shelf Life, sealed containers:	12 months	12 months	-

MIX RATIOS

	Parts By Weight	Parts By Volume
Part A:	100	1
Part B:	90.6	1

CURE SCHEDULE

Recommended Cure: 1 day at 77°F

* Typical cured properties were determined using the recommended cure schedule. Some differences in properties may occur with alternate or other cure schedules.

TYPICAL CURED PHYSICAL PROPERTIES

Hardness, Shore D [24 hr @ 77°F]:	55 - 65		
Lap Shear Strength, psi Al/Al:	@-40°F: 2,535	@77°F: 1,920	@179.6°F: 340
T-Peel Strength, pli Al/Al:	@-40°F: 5	@77°F: 6	@179.6°F: 6
Temperature Range:	-40°F to 149°F		

STORAGE AND HANDLING

Store indoors in a warm (80°F-90°F), dry environment. This product is highly moisture-sensitive; keep containers sealed when not in use. After dispensing, blanket with nitrogen or dry air and reseal promptly. For industrial use only—handle with care. MSDS is provided with samples and available upon request. Prolonged skin contact may cause rash or dermatitis. Always use eye protection and appropriate gloves. If ingested, seek medical attention immediately.

DISCLAIMER

The information provided is based on our research and is considered accurate. However, no warranty, expressed or implied, is given.

**Look for other adhesives
in the Farrier's Choice line!**



Directions:

1. To open cartridge turn retaining nut 1/4 turn counter-clockwise.
2. Place cartridge into applicator gun, dispense material until two pistons are level or equalized.
3. Affix static mixer by aligning keyed interface. Use retaining nut to securely fasten mixer into cartridge.

Preserving unused product:

Replace dual plug and secure with cap.

Dispensing gun and mix tips (sold separately)

Use to dispense Farrier's Choice adhesives. Also works with other products that come in 210mL cartridges.



**For quality and consistency always use
original Quadro mixers!**



Distributed Exclusively by Castle Plastics
Supporting the hoof care industry for over 25 years.
978-534-6220 • farrierschoice.com



Made in the USA by Adhesive Engineering