

PACK-SEAL SOFT

Superior Cushioning & Flexibility

A fast-curing, urethane-based adhesive offering full sole protection, cushion, and support.



Easy-to-use solution designed to provide comfort and protection in every step.



Set Time	Base Material	Packaging	Application
30 seconds	Urethane	Durable 210mL cartridge	Packs, protects, cushions, provides comfort and support

Farriers Choice® Pack-Seal Soft is a fast-setting, two-component polyurethane adhesive that provides cushioning and lasting flexibility. It cures soft, ideal for thin-soled horses or those needing extra comfort. This odorless, non-sag formula bonds to the hoof and optional rubber pads, sets in 30–40 seconds, and is weight-bearing immediately.

- **Flexible & Strong** – Provides a cushioned, protective layer without compromising strength.
- **Quick Set Time** – 30-second cure time ensures fast and efficient application.
- **Superior Cushioning** – Absorbs impact and shock, reduces stress on the hoof while providing comfort.
- **Versatile Application** – Adheres securely to the sole and frog, works with or without pads.
- **No Mess, No Waste** – Resealable, heavy-duty cartridge prevents leaks, preserves product, and allows multiple uses.

Resealable twist-off cap protects product between uses.

*Cap and mixing tip fit into the cartridge **one way only**—to prevent cross contamination and eliminate material waste.*

Mixing tip included with each cartridge!



Trusted by Farriers, Proven in the Field. Learn more at farrierschoice.com.

Farrier's Choice® offers a range of specialized adhesives, each tailored to address specific hoof care needs and designed to meet specific challenges faced by professional farriers, ensuring optimal hoof health and performance.



Farriers Choice® Pack-Seal Soft

Similar to our Pack-Seal Blend, but cures to a soft consistency, maintaining flexibility. Provides superior cushioning, and ideal for addressing issues with thin-soled horses or those requiring a softer, more flexible support material.

Set Time: 30 seconds **Base:** Urethane **Color:** Clear

TYPICAL PROPERTIES

	Part A	Part B	Mixed
Color:	Amber	Light Amber	Amber
Viscosity @ 77°F, cps:	1,500 - 3,500	850 - 1,700	Paste
Weight per gallon, pounds:	8.7 - 9.4	8.10 - 8.70	-
Gel Time, [9.4 grams @ 77°F], seconds:	-	-	30 - 40
Shelf Life, sealed containers:	12 months	12 months	-

MIX RATIOS

	Parts By Weight	Parts By Volume
Part A:	100	1
Part B:	94	1

CURE SCHEDULE

Recommended Cure: 1 day at 77°F

* Typical cured properties were determined using the recommended cure schedule. Some differences in properties may occur with alternate or other cure schedules.

TYPICAL CURED PHYSICAL PROPERTIES

Hardness, Shore A [24 hr @ 77°F]:	30 - 50
Temperature Range:	-40°F to 149°F

STORAGE AND HANDLING

Store indoors in a warm (80°F-90°F), dry environment. This product is highly moisture-sensitive; keep containers sealed when not in use. After dispensing, blanket with nitrogen or dry air and reseal promptly. For industrial use only—handle with care. MSDS is provided with samples and available upon request. Prolonged skin contact may cause rash or dermatitis. Always use eye protection and appropriate gloves. If ingested, seek medical attention immediately.

DISCLAIMER

The information provided is based on our research and is considered accurate. However, no warranty, expressed or implied, is given.

*Look for other adhesives
in the Farrier's Choice line!*



Directions:

1. To open cartridge turn retaining nut 1/4 turn counter-clockwise.
2. Place cartridge into applicator gun, dispense material until two pistons are level or equalized.
3. Affix static mixer by aligning keyed interface. Use retaining nut to securely fasten mixer into cartridge.

Preserving unused product:

Replace dual plug and secure with cap.

Dispensing gun and mix tips (sold separately)

Use to dispense Farrier's Choice adhesives. Also works with other products that come in 210mL cartridges.



*For quality and consistency always use
original Quadro mixers!*



Distributed Exclusively by Castle Plastics
Supporting the hoof care industry for over 25 years.
978-534-6220 • farrierschoice.com



Made in the USA by Adhesive Engineering